

Element AIM Software Update Version 1.1.114



Dear customer,

We have the pleasure to announce you that your new Element AIM software update (Vers.1.1.114) is available and includes the following changes:

- **Bounding box feature available for both fecal and urine analysis**
This feature will help you to understand what the Artificial Intelligence is focusing on during its classification and will allow you to have a faster image review process. After double clicking a specific image, you now have the option to click the “Target” button in the upper left corner of the image. As you can see below, the “Target” button will turn orange when it is on and showing the orange “Bounding box” surrounding the object.



- **Fecal analysis:**
 - **Quick scan feature available**
This feature is now also included for the fecal analysis and will dramatically reduce the time to complete a fecal test (6.5 minutes only).
 - **QR Code for the fecal preparation available**
A new QR Code can be scanned before a fecal analysis to help you in the preparation of the sample.

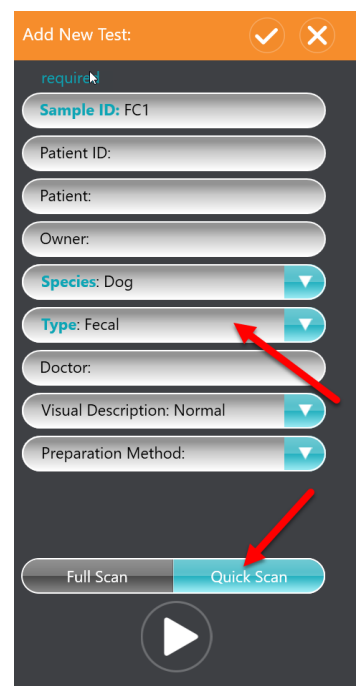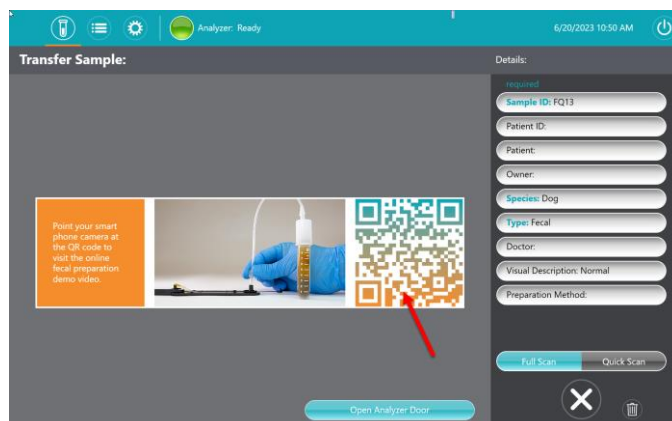

- **Artificial Intelligence significantly improved for fecal analysis**
The sensitivity, the specificity and the quantitation accuracy have been increased for the classification of the items.
- **Sending of a favorite picture to your PIMS**
You can select a picture as a favorite to send it preferentially to your PIMS.
- **Dilution guide available**
You can now access to the dilution guide directly on the *Settings* screen.
- **Enhanced sorting of displayed top 40 highest confidence images**
This ensures that the best images are at the top of the list.
- **Updated translations for different languages**

This new software update is available today. As with other updates, your AIM app will prompt you with an alert letting you know that a new update is available.

PLEASE NOTE: Any system not connected to the Internet will not receive the auto-update. If you did not get an update alert on your AIM app, please simply follow the outlined steps in the attached update instruction.

In case of questions concerning the software update, please contact your local scil animal care company service hotline.



Sincerely,
your scil animal care company

Product manager laboratory diagnostic

Element AIM Update Instructions


If the Element AIM is connected to the internet, your AIM app will prompt you with an alert letting you know that a new update is available.




PLEASE NOTE: Any system not connected to the Internet will not receive the auto-update. If you did not get an update alert on your AIM app, please simply follow the outlined steps to connect the Element AIM to the network and /or update the software manually.

- **How to check if the AIM Computer is connected to the Internet**


1. Go to *Settings*, activate the “Administrator” button.

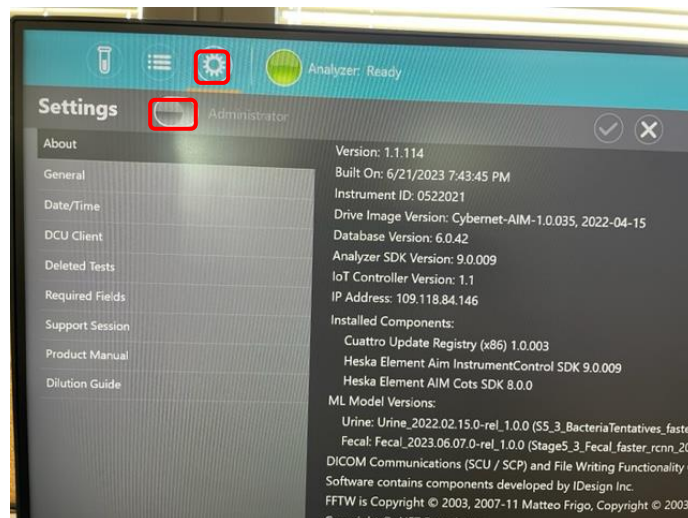


2. Enter for username: “AIM” and password: “AIM” and press the checkmark button.

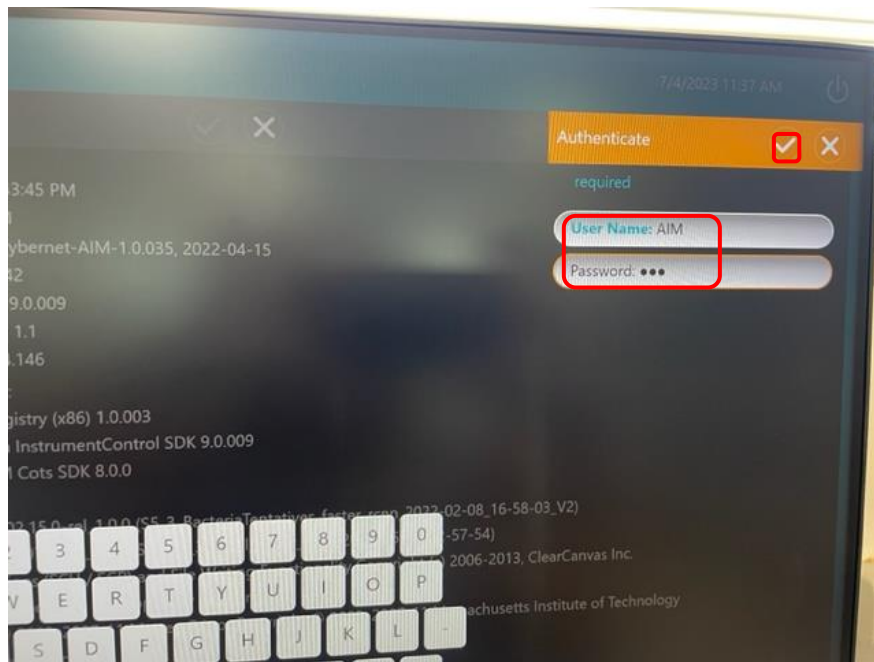

3. Press the “*Update Application*” button.

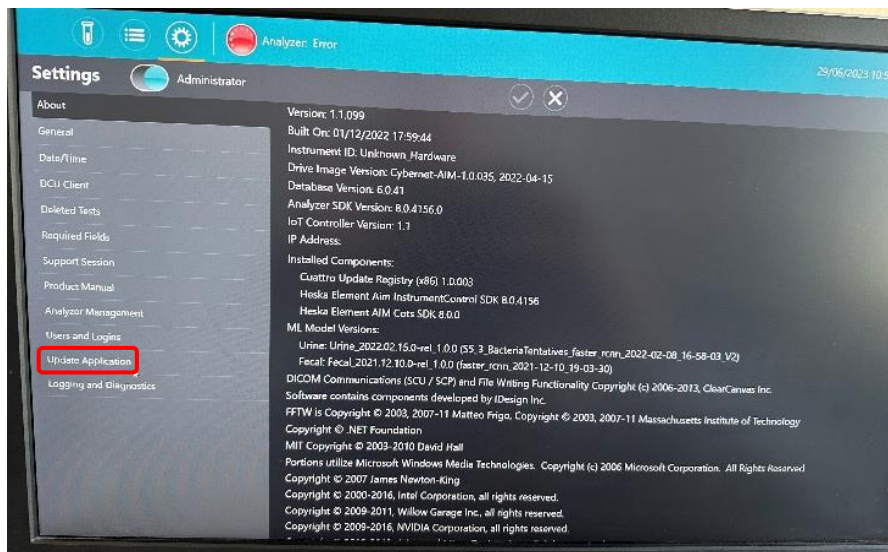

4. If the “*Check for Update on Server*” is highlighted, the AIM Computer is connected to the Internet. The update will be downloaded automatically and you will receive a message that a new update is available after the download is finished.

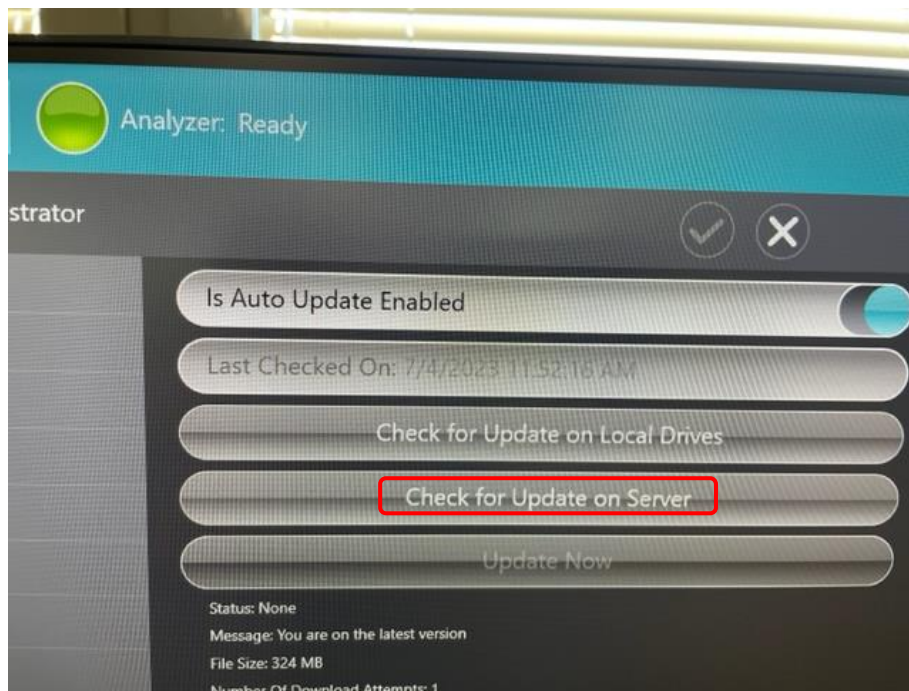

5. If the “*Check for Update on Server*” button is not highlighted and the status shows “*Error*”, the AIM computer is not connected to the internet, please contact the Technical Service.